


**FITTING GUIDANCE**

0.1 CENTRE Bed Board to elbow

1.1 SPLIT Device


1.2 POST Double-velcro end

1.3 WRAP Target arm

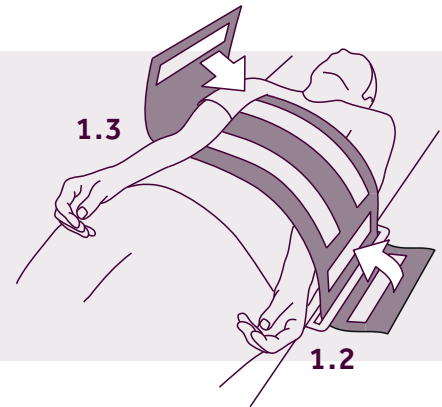

2.1 HANG Thumb Cushion (PAD DOWN)

2.2 ATTACH Thumb Strap

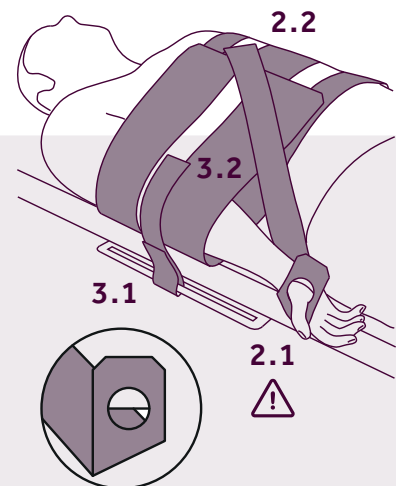

*OPTIONAL - Retention Strap*

3.1 *POST Double-velcro end*

3.2 *SECURE Strap*

[DRAPE FOR PROCEDURE]

4.1 REMOVE Straps

[FIT COMPRESSION BAND]

4.2 RE-WRAP Arm Support (LOWEST FITTING)

4.3 SLIDE to forearm

4.4 SPLIT Centre perforation

4.5 PLACE Over Patient Head

4.6 *OPTIONAL - TEAR/FOLD Neck protector*

4.7 FOLD/SECURE at shoulders

